Product code: 55791

Digital alcohol tester - with 5 nozzles

## USER'S GUIDE

## Product description:

With this digital alcohol probe you can easily and quickly check the presence of alcohol in your body! Thanks to its compact size, you can carry it anywhere, even in your pocket, and with its handy key ring you can use it as a key ring. Immediately after use, the probe analyzes and displays the measured blood alcohol level on the digital display. You can prevent drunk driving and drunk and accidents with the alcohol probe

- Compact size
- Automatic measurement preparation
- Instant analysis
- LCD display with backlight
- Gray color with key ring
- Auto power off
- Power-saving, battery-operated
- 5 plastic nozzles included

## Device parts:

1. Nozzle
2. Power Button
3. LCD monitor
4. Battery cover
5. Key ring

## Startup:

Open the battery compartment on the back of the device and insert 2 AAA (LR03) 1.5V batteries. Pay attention to correct polarity when inserting batteries. Replace the battery compartment cover. It is recommended that you remove the batteries when not in use for a long time. The device can be operated from a rechargeable battery with the same parameters. The display's backlight flashes to indicate low battery charge level of the batteries.

## To use the device:

1. Preparation (Heating)
  - Press the power button until you hear a beep. Briefly press the power button to turn on the display backlight only for the duration of the button press. (This can be useful in the dark, for example if you want to use it as a key chain).
  - When on the display "Warm -up" is displayed in a countdown (10-0) when the counter reaches zero, the device is ready for operation.
  - Note: If the preparation process fails, the alcohol probe will automatically restart and restart 1-3 times. Initial use and longer periods of non- use may require cleaning and preparation of the sensor. In this case, press the power button repeatedly until the device is ready for measurement.
2. Alcohol level measurement
  - "Blow" appears on the display to indicate that the preparation was successful and the device will count down to 10-0. During the countdown blow into the mouthpiece of the product continuously, with a constant and uniform air flow. There is no need to put the nozzle in the mouth, just close to the mouth.
  - For successful measurement, blow into the nozzle for 3-5 seconds. After the countdown, the analysis of the measured

values begins and the result appears on the display.

- "Caution" will be displayed if the reading is between 0.02% (0.2 g / l) and 0.05% BAC (0.5 g / l).
  - "Danger" will be displayed if the measured value is equal to or greater than 0.05% BAC (or 0.5 g / l).
3. Auto power off
    - The unit automatically switches off after ~ 10 seconds after the measurement, ensuring long life and environmentally conscious operation. If the preparation failed, the device will turn off automatically.
- Note:
- It is the manufacturer's recommendation to double-check your blood alcohol level 2-3 times in a row to make sure it is accurate.
  - Blinking of the display backlight may indicate that the batteries are low.

## Warnings:


- We recommend that you wait 20 minutes after drinking alcohol to check the alcohol level, as alcohol will be absorbed into the digestive tract and only then will it be detectable and the alcohol remaining in your mouth will take time to evaporate.
- Wait at least 3 minutes between two measurements to obtain an accurate result.
- Do not smoke or eat any food for 20 minutes between drinking and measuring.
- Do not use chemicals to clean the device.
- Do not spray liquid into the nozzle!
- Do not use paint, insecticides or other alcoholic substances near the appliance.
- Do not expose the device to contaminated air as this may affect the measurement.
- The measurement results of this alcohol probe are for reference only and the manufacturer and the distributor accept no liability whatsoever.

## Proclamation!

The air-based alcohol tester should only be used to indicate the presence of alcohol in the air or blood. It must not be the device to determine the toxicity degree or if it's safe to drive, handle machinery or perform hazardous activities.

## Specification:

Measuring range:	0.00 - 0.19% BAC (blood alcohol level) or 0.0 - 1.9 g / l
Accuracy:	0.01% BAC (blood alcohol level)
Power supply:	2 x 1,5 V AAA (battery not included)
Display:	1.14" LCD
Backlight:	Orange, Automatic
Material:	Plastic
Size:	114 x 70 x 29 mm
Color:	Gray
Weight:	70 g

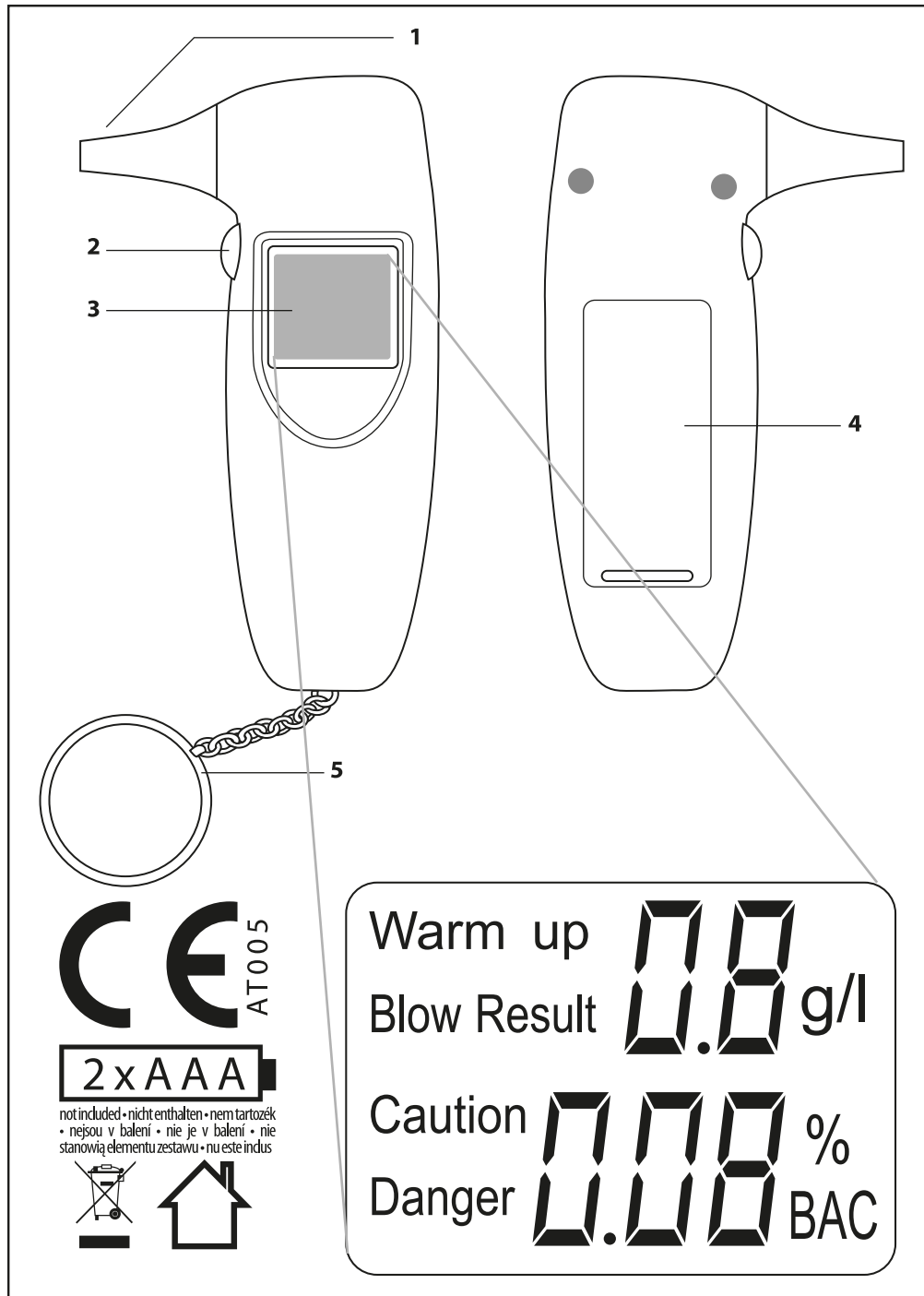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